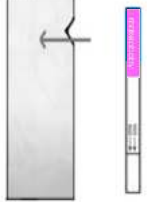



Test Instructions


Please read ALL the instructions before opening or using the test.

If you need more information, please contact us at support@niagarareproductivejustice.com


- 1 Remove the test from the package by tearing at the notch.


- 2 Holding the strip by the coloured end, dip the strip into the sample up to the max line.

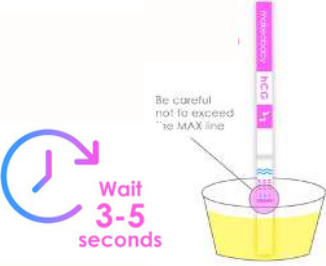
- 3 Lay the strip flat on a non-absorbant surface.

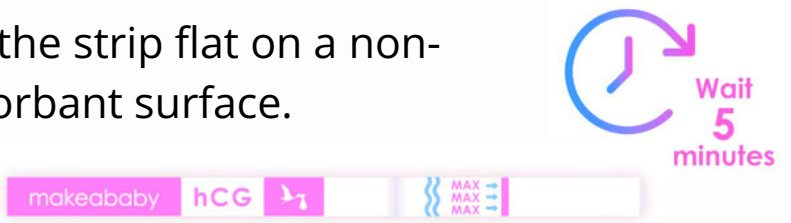
- 4 Read results within 30 minutes. Positive results may show immediately, negative results can be read in 5-10 minutes.

RESULT INTERPRETATIONS

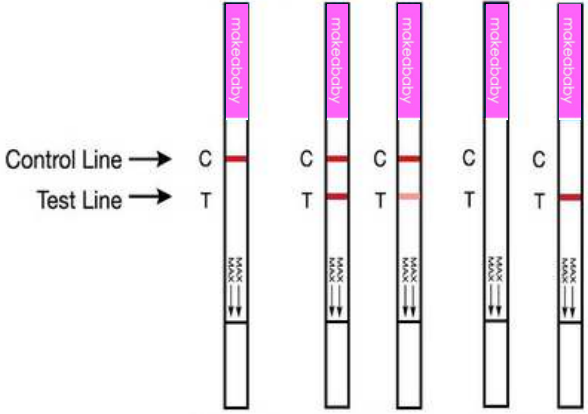

Negative
A pink colored band appears only at the control region.

Positive
A clear pink control band and a detectable test band appear. This indicates pregnancy.

Invalid
No visible band at the control region. Repeat with a new test device.

Negative Positive Invalid